



mlock

The MLOCK works by holding your chair still while you transfer in or out of it. This is particularly useful for people using a wheelchair or those with other restricted mobility issues.



The MLOCK uses air cell technology with the effect of forcing the stabilising antislip feet onto the floor, securing the chair. The air cell is inflated by means of a battery operated Mini-pump (or Manual Hand pump) which can be situated almost anywhere on the chair. An air release button allows the feet to retract and the chair to become mobile once again on the castors.

To operate:

- 1) To Lock: Inflate the air-cell to activate the MLOCK by pressing the '+' button on the battery hand-pump for approximately 8-10 seconds (or by repeatedly squeezing the manual pump bulb) until the non-slip feet are firmly in contact with the floor, stabilising the chair.
- 2) To Unlock: Press the '-' button on the battery powered hand-pump (or by pressing the air release valve on the manual hand pump) to retract the non-slip feet and allow the chair to freewheel once more.*

*Please note that the air slowly releases over a period of time on the MLOCK.



miniPump™ battery powered option (4 x AA batteries required)

Notes:

The MLOCK feet settings are adjustable dependent on the type of floor surface. Before the first use or if you change the floor covering the MLOCK is used on, set up and adjustment is advised to ensure that the non-slip feet are set at the correct level before activation.

A good way to set the chair up is to put a coaster underneath the feet and set the height.

Get in touch

01454 329210

sales@ergochair.co.uk

ergochair®